

Installation Guide for Data Port Cable (Drivers)

The Data Port Cable will allow the Sure Classic, Sure Edge and Vocal Point users to download their stored memory readings from their respective meters utilizing the Specialty Medical Diabetes Management Software.

The data port cable plugs directly into your computer's USB port which will allow the transfer of your meter's memory into the Data Management Software.

It might be helpful to print these instructions out prior. To install:

1. **Click** on the software link *"USB Adapter Drive"* to start this process.
2. A pop up window will appear asking user to Run or Save this file. **Click "RUN"**. A progress screen will appear. This download should take approximately 30 seconds.
3. A pop up screen will appear asking the user *"The publisher could not be verified, are you sure you want to run this software?"* **Click "RUN"**. At this point a *"Windows Installation Wizard"* screen will pop up informing the user that installation is in process.
4. A pop up screen will appear showing the user that installation is complete. **Click "FINISH"** to exit the program.