

## Downloading Instructions Sure Classic, Sure Edge and Vocal Point Diabetes Management Software

Prior to downloading please verify that your computer has the required memory to install this program. This will utilize 18.1 MB of your computers memory. Also you will need to have a high speed internet connection. A "Dial-Up" internet connection will

**Please Note:** The following address will be required. Please cut and paste onto your web browser.

[http://www.specialtymedicalsupplies.com/prod\\_sureedge\\_software.php](http://www.specialtymedicalsupplies.com/prod_sureedge_software.php)

This will bring you into our webpage for the free software download. You will need to download **(2) Two files**. The following instructions are for the ***Specialty Medical Diabetes Management System***. The additional software file is the **"USB Drivers"** for the data port cable that is required to connect your meter to the computer. Please see 2<sup>nd</sup> attachment for the instructions.

1. Click on the link ***"SureEdge Diabetes Management System"*** to initiate the installation.
2. A pop up menu will appear asking *"Do you want to run or save this file?"*; **Click "RUN"**  
The program will commence to download taking approximately 3-8 minutes depending on the speed of your computer.
3. A pop up menu will appear stating *"The publisher could not be verified", do you want to run this software?"* **Click "RUN"**
4. Immediately a Windows Screen will appear labeled *"Self Extracting Archive"*; **Click "INSTALL"**
5. After completion, this will load the program into a folder on your desktop labeled *"Specialty Medical Diabetes Management System"*. Please go to your computers desktop and **"Double Click"** on this folder's icon.
6. A pop up window will appear displaying 3 icons. **"Double Click"** on the *"Specialty Medical Diabetes Management System"* icon.
7. A windows "Installer Set Up Wizard" will appear; **Click "NEXT"**
8. A "Select Installation" menu will appear; **Click "NEXT"**
9. A "Confirmation Installation" screen will appear; **Click "NEXT"**. At this point a progress window will appear. This should take less than 1 minute.

10. An “Installation Complete Screen” will appear; **Click “NEXT”**. This screen will disappear after finishing (less than 30 seconds).

11. Exit or close out of remaining folder by clicking on the **X** in the upper right hand corner.

12. At this point a new icon will appear on your desktop labeled “*Specialty Medical Diabetes Management System*”. This icon will appear with a picture of a red heart. You may now double click on this icon to enter the program.

## **Installation Guide of SureChek Data Port Cable (Drivers)**

The Data Port Cable will allow the SureChek user to download their stored memory readings from their SureChek Blood Glucose Meter utilizing the SureChek Diabetes Management Software.

The data port cable plugs directly into your computer's USB port which will allow the transfer of your SureChek Meter's memory into the Data Management Software.

1. **Click** on the software link "*USB Adapter Drive*" to start this process.
2. A pop up window will appear asking user to Run or Save this file. **Click "RUN"**. A progress screen will appear. This download should take approximately 30 seconds.
3. A pop up screen will appear asking the user "*The publisher could not be verified, are you sure you want to run this software?*" **Click "RUN"**. At this point a "*Windows Installation Wizard*" screen will pop up informing the user that installation is in process.
4. A pop up screen will appear showing the user that installation is complete. **Click "FINISH"** to exit the program.