

Instructions for downloading Drivers for "Vista" labeled cables

1. After clicking on "Download Driver", you will see a Pop-Up Screen (P/U Screen) labeled "File Download" with options: "Run, Save, Cancel"

Click "RUN"

P/U Screen will appear "Downloading will Occur"

2. When downloading finished, you will see a P/U Screen "The published could not be verified", with options "Run or Don't Run"

Click "RUN"

3. P/U Screen will appear "InstallShield Wizard for Silicon Laboratories CP210X Driver for Windows XP/2003 Server/Vista/7 v.6.00"

Click "Next"

4. P/U Screen will appear "Licensing Agreement"

Click "Accept" in "Radio Button" options, and then click "Next"

5. P/U Screen will appear "Choose Destination Location"

Choose and Click "Next"

6. P/U Screen "Ready to Install Program" will appear. Click "Install" (Status window will appear showing progress).

7. P/U Screen will appear "InstallShield Wizard Complete"

Click "Finish"

8. P/U Window will appear "Silicon Laboratories CP210X USB to UART Bridge Driver Installer" with destination address also.

Click "Install"

9. P/U Screen will appear "You must restart your computer. Do you want to restart your computer now?"

Click "Yes"

10. This process will install the driver for your Cable labeled "Vista"

ERROR: undefined
OFFENDING COMMAND:

STACK: